

6. Charging the HOTPOD hot pack should only be done by adults.
7. Do not leave the hot pack alone whilst charging-monitor at all times as per instructions.
8. HOTPOD must be charged using the charging arm provided, and is not to be tampered with or repaired unless certified by McGloins-Supertex Pty Ltd.
9. If the cord is damaged it must be replaced by the manufacturer, its service agent or similarly qualified persons to avoid hazard and misuse.
10. Do not place anything on or around the hot pack whilst charging.
11. If the product swells during charging, remove from charger and lightly circulate the contents of the pack, if swelling persists, do not use.
12. Do not disconnect the charging pack with wet hands.
13. Do not wash with any substance or submerge in water/liquid- use a soft damp cloth and gently wipe.
14. Do not allow any sharp objects to come into contact with the pack, if signs of leakage, abnormal noise or smoke are apparent - do not use.
15. Under no circumstance should the HOTPOD be used as a cushion and sat upon.
16. Use with caution, if pack seems too hot, wait for it to cool prior to use.
17. If you currently use pain medication, use the HOTPOD hot pack with caution as your sensitivity to heat may be reduced.
18. The hot pack is not intended for medical use.
19. Avoid prolonged direct contact with skin; do not use if you have sensitivity to heat.
20. When storing the appliance, allow it to cool down completely first.
21. Read and retain instructions for future use.

Symbol Key:

- Read operator manual



- Do not insert sharp objects



- Do not wash



- Pack may get hotter than expected. Use with caution.



User Manual

⚠ WARNING

<p>MUST CHARGE ON A FLAT SURFACE</p>	<p>DO NOT USE WHEN CHARGING</p>
<p>DO NOT CHARGE ON UPRIGHT OR INCLINED</p>	<p>DO NOT INSERT PINS</p>

HOTPOD Electric Hot Pack

- 1.HOTPOD ELECTRIC HOT PACK comes with a 12-month warranty, provided it has been used and stored as recommended. Should you require any assistance with HOTPOD ELECTRIC HOT PACK, please contact McGloins-Supertex Pty Ltd Head Office on 02 9836 1000 or customerservice@mstx.com.au. Our Customer Service consultant will assist you with your enquiry.
- 2.HOTPOD is a portable, multipurpose electric hot pack for use on the human body, recommended from 5yrs +.
- 3.Charged in 10-15 minutes and provides up to 8 hours of heat, depending on environmental conditions.
- 4.Charging the HOTPOD hot pack **must be** with adult supervision.
- 5.Thermo-regulator temperature controller automatically switches off between 50-60 degrees Celsius.
- 6.The triple permanent sealed bag ensures zero leakage; and no need for refilling.

HOTPOD Electric Hot Pack Features

- 1.Great portable pack for all over body use for aches, pains or warmth especially during the cooler months, charged in 10-15 minutes lasting up to 3-4 hours.
- 2.HOTPOD is the next generation hot water bottle, an electrically heated pack with a thermo-regulator that automatically stops heating once it reaches the programmed temperature.
- 3.When charging, the pack may generate vaporised gas and cause the bag to swell; simply shake the pack lightly applying light pressure to release the gas that may have built up.
- 4.The ergonomic and durable design of the charging arm acts as an extra precaution and regulator should the pack swell due to vaporised gas. In this instance the arm has an extra sensor and will click and shut off.
- 5.HOTPOD is manufactured with a heating element preventing overheating and offering the latest technology for electric hot packs.
- 6.LED charging light goes out when the bag is ready for use.
- 7.RATED VOLTAGE: 220-240V~
- 8.FREQUENCY: 50Hz
- 9.AVERAGE POWER: 360W

HOTPOD Electric Hot Pack Instructions

1. Check for any signs of damage or leaking before each use - do not use if apparent.
2. Before charging, remove cover, place the hot pack and charging arm flat on an even, stable, heat-proof, flat surface with the plug facing upward.
3. Insert the charging arm with the hot pack in between each arm; connect to the plug and power. Note the LED light turns 'on' when charging. **Do not use hot pack whilst charging.**
4. If the pack swells during heating, simply apply minimal pressure to the pack whilst gently circulating the contents- if the pack does not decrease in size i.e. the gas is still swelling the pack, remove from power source immediately and allow to cool. A small amount of noise while heating is normal.
5. Once the unit has reached programmed heat, the LED light will switch off and the hot pack is ready to use.
6. Gently manoeuvre the pack to effectively circulate and disperse the heat before use.
7. If there is a possibility that prolonged contact with the skin should occur, an adequate cover should be used to prevent burns.
8. If you are using a HOTPOD with a removable cover, ensure you put the cover back on completely after charging and before use.

HOTPOD Electric Hot Pack Cautions

1. Check the product before each use; do not proceed if damage is apparent. Replacement of Hot Pack is recommended every 1-2 years, depending on usage.
2. Must charge on a flat surface removed from cover; **NEVER use hot pack whilst charging.**
3. Do not charge on a bed or any heat sensitive surface; do not charge the unit facing down; always keep flat with the plug facing upwards.
4. HOTPOD is recommended for ages 5yrs + and is not intended for use by persons (including Children) with reduced physical sensory or mental capabilities or lack of experience or knowledge unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
5. Keep out of reach of children; ensure children do not play with the appliance. An adult must supervise use by children at all times.